

**Tufts Medicine Care at Home
Annual Skills
CoaguChek XS System**

Employee: _____

Date: _____

Evaluator: _____

Date: _____

Purpose: *To verify that the clinician demonstrates correct observable technique and infection control practices while using the CoaguChek XS System. Verbalizations and technical troubleshooting are evaluated separately.*

Step	Key Actions / Critical Behavior	Met	Needs Improvement	Comments
1	Assembles CoaguChek XS meter, test strip, and necessary supplies			
2	Performs hand hygiene, dons gloves, prepares lancet device, powers on meter			
3	Inserts test strip correctly; waits for blood drop prompt			
4	Uses proper capillary sampling technique with lancet			
5	Applies blood to test strip correctly within time window			
6	Reads and records test result as displayed			
7	Disposes of used lancet and strip in appropriate sharps/biohazard containers			

Evaluator Summary:

Competent

Repeat Evaluation

Evaluator Signature: _____

Date: _____

Employee Signature: _____

Date: _____