

CoaguChek XS Test

NAME: _____ DATE: _____

1. After removing the test strip from the container, it is important to close the cap tightly:
 - a) True
 - b) False
2. Sample must be applied to the test strip within ten minutes of removing the strip from the container:
 - a) True
 - b) False
3. INR is a reporting format that stands for International Normalized Result:
 - a) True
 - b) False
4. Every time a blood test is performed, the meter also performs a built-in quality control test:
 - a) True
 - b) False
5. If the built-in quality control test fails, the meter will still give a test result:
 - a) True
 - b) False
6. The most recent patient result appears first when reviewing memory:
 - a) True
 - b) False
7. Test strips expire 18 months after opening the bottle or on the expiration date listed on the bottle (whichever comes first):
 - a) True
 - b) False
8. You have to get blood sample onto test strip within 30 seconds:
 - a) True
 - b) False
9. What could cause the word "Code" to appear on the display?
 - a) Code chip is damaged
 - b) Code chip is missing
 - c) Code chip is not snapped firmly into place
 - d) All of the above
10. What could cause "QC" to appear on the display?
 - a) The test strip failed the internal quality control check.
 - b) The test strip is unusable
 - c) All of the above